Document de bon usage patient

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Proper patient use document

BREATH TEST WITH HELIKIT® 75 mg

UNDERSTANDING YOUR DRUG PRESCRIPTION FROM YOUR DOCTOR

What is the purpose of this functional breath test?

Your doctor has prescribed you HELIKIT®:

- For **diagnostic purposes**, to **check** for bacteria called *Helicobacter pylori* in your stomach.
- For **monitoring purposes**, to **confirm that *Helicobacter pylori* has been eliminated** after your antibiotic treatment. This is because there is a risk that the antibiotic treatment may fail. It is therefore necessary to check whether the bacteria have been eradicated.

Use the **first prescription** to buy the test **from a pharmacy**. Give the **second prescription** to the **medical laboratory** to carry out the test.

WHAT YOU NEED TO KNOW BEFORE TAKING THE TEST

Before going to the laboratory, you must follow these instructions:

- **Fast** from the evening before the test – this means you should not drink, eat or smoke
- Stop using all antibiotic treatments **at least four weeks** before the test
- Stop using drugs that reduce acid production in the stomach (proton pump inhibitors) **at least two weeks** before the test

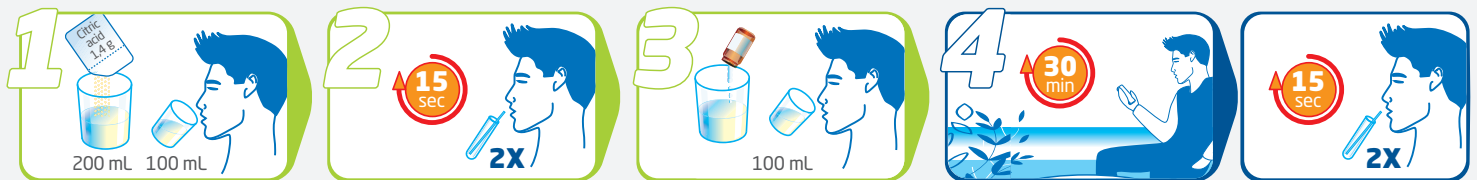
Where should the test be performed?

The HELIKIT® test must be done **at a laboratory**. Remember to book an appointment before you go.

HOW IS THE TEST DONE?

HELIKIT® powder for oral solution is administered **orally**. This **quick, non-invasive** breath test is intended for **adult use only**.

Protocol for sample collection at the medical laboratory:

Step 1: First, you will drink 100 ml of a starting volume of 200 ml of a citric acid solution.

Step 2: Next, take a deep breath in and then breathe out through a straw into two tubes for 15 seconds each. Keep breathing out while the healthcare professional removes the straw from the tube.

Step 3: Next, drink the remaining 100 ml of the citric acid solution, into which labelled urea will have been dissolved.

Step 4: Repeat step 2 after 30 minutes.

This sheet is a supplement to what you have discussed with your doctor and does not replace your doctor's opinion. If you are unsure of anything, ask your doctor or pharmacist for advice.

Before doing the test, carefully read the package insert found inside the box or available on the public database of medicinal products: <http://base-donnees-publique.medicaments.gouv.fr/>.

Report any suspected adverse drug reactions immediately to your Regional Pharmacovigilance Centre or by going to <https://signalement.social-sante.gouv.fr/>.

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