caya® diaphragm gel

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Applikator: Hubert de Backer n.v.
9100 Sint-Niklaas, Belgium

caya® diaphragm gel

Instructions for use
(Please read carefully and keep for future reference.)

Caya® is a contraceptive gel for use with natural latex rubber or silicone diaphragms and cervical caps.

Application

Before inserting into the vagina, place about one teaspoonful (about 4 ml) of Caya® Gel in the diaphragm or cervical cap. Then follow the instructions for inserting the diaphragm or cervical cap. The cellulose gel creates an additional mechanical barrier between the diaphragm (or a cervical cap) and the uterine orifice. The lactic acid with its physiological pH of 3.8 has a motility inhibiting effect on the sperms - they are slowed in their forward movement or stopped.

Composition


Water, lactic acid, sodium lactate, cellulose, sorbic acid, vanillin.

Storage


Store in a dry place between 4 °C and 25 °C. Always keep the tube closed. For minimum shelf-life, see printing on tube.

Use within 3 months


of opening.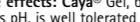


DeltaMed GmbH
Raiffeisenstr. 8a,
61169 Friedberg, Germany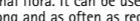


0124 medical device
Made in Germany

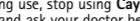
Vertrieb: KESSEL medintim GmbH
Kelsterbacher Str. 28
64546 Mör./Walldorf Germany
Telefon: +49 6105 20 37 20
www.medintim.de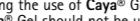

Distributor Australia:
Medical Industries Australia
87 Mars Road
Lane Cove NSW 2066
T: +61 2 9490 6200
E: sales@medintim.com.au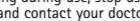

EAN 4013273001250

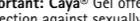Applicator:
Hubert de Backer n.v.
9100 Sint-Niklaas, Belgium

Important: Caya® Gel offers NO protection against sexually transmitted diseases. It is only effective when used together with a diaphragm (e.g. Caya® contoured diaphragm) or a cervical cap. Caya® Gel IS NOT a safe contraceptive

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